



Oxivir® Excel Wipes

1 WIPE - 1 MINUTE



**One-Step Hospital Grade
Disinfectant Cleaner**
Based on Accelerated Hydrogen
Peroxide (AHP®) Technology to deliver
fast, broad spectrum disinfection with
enhanced cleaning power.




AUST L 343051

How to Use Oxivir® Excel Wipes



Remove one wipe from pack.
Close Lid.

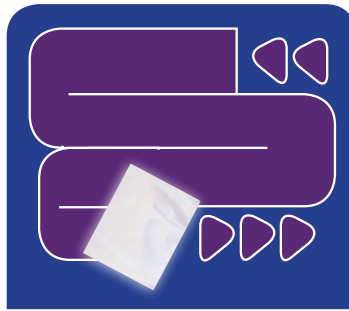

Work from clean to dirty in an S shape.
Allow the surface to air dry



Dispose used wipes into a suitable waste bin

READY TO USE PRE-WETTED CLEANING AND DISINFECTING WIPE

Hospital Grade Disinfectant

ARTG No: AUST L 343051 - Oxivir Excel Wipes

Active: Hydrogen Peroxide 0.36%w/w

EFFICACY DATA:

When used as directed, Oxivir Excel Wipes are effective against a wide variety of pathogenic micro-organisms including Viruses, Bacteria and Yeast.

ENVIRONMENTALLY RESPONSIBLE:

Active ingredient breaks down to oxygen and water.



PRODUCT

Oxivir Excel Wipes
24 x 100 wipes
(27 x 20cm wipe size)

CODE

70024385

Bacteria and Yeast Contact Time

Staphylococcus aureus ... 30 sec*
Enterococcus hirae 30 sec*
Pseudomonas aeruginosa.. 30 sec*
Candida albicans.....1 min*

Viruses Contact Time

Herpes Simplex Type 11 min#
Murine hepatitis virus (MHV) -11 min#
(^surrogate to make COVID-19 claim)
Adenovirus Type 5.30 sec**
Poliovirus Type 130 sec**
Norovirus30 sec**

** Test Method (dirty conditions) EN14476 # Virucidal Carrier Test

*Test Method (dirty conditions) EN13727 Yeast (Candida albicans) Test Method (dirty conditions) EN16615

^ <https://www.tga.gov.au/surrogate-viruses-use-disinfectant-efficacy-tests-justify-claims-against-covid-19>



Contact our friendly Customer Service team today

AUSTRALIA - 1800 647 779 - diversey.com.au

NEW ZEALAND - 0800 803 615 - diversey.co.nz

DISCLAIMER: Please check local regulations and guidelines for any references provided. These materials are provided for general information purposes only and do not replace each user's responsibility to assess the operational, legal and other requirements applicable to each facility. May not be copied or distributed without prior written permission of Diversey. © Diversey 2020 All rights reserved. AHP® and Design and Oxivir® are trademarks of Diversey, Inc.